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ENGLISH

HOW TO USE

1. Check if the stethoscope is assembled.
2. Put the eartips.
3. Put the diaphragm where you want to hear.
4. This stethoscope is indicated only to a qualified professional.

Rappaport Model 200

The Rappaport model is very practical. Designed with 5 different sized chestpieces, this instrument is highly useful in the detection of a wide range of heart and lung sounds. With such a wide range of uses you find that you will be successful in your needs.

Components:

Diaphragm:

Use the diaphragm to detect faint high-pitched respiratory sounds and components of certain cardiac sounds. The diaphragm attenuates low-pitched frequencies making high-pitched frequencies easier to be recognized. To accommodate a range of patients, use the various sized diaphragms to better isolate the origin of sounds.

Bell:

You can match the bell size to the characteristics of your patient to better locate the origin of a sound. For this reason there are three different sized bells: small, medium and large

Accessories:

A pair of diaphragm in adult and child sizes
Three bells in small, medium, and large
Two pair of eartips

GUARANTEE

1. This stethoscope is guaranteed for one year from the purchasing date and this warranty is effective only by presenting the **RECEIPT OF PURCHASE** stating the buyer's name, purchasing date, name and address of the distributor; and buyer should always present the receipt every time technical service is needed.
2. The guarantee does not cover the cost of round-trip transport for repairs, the result of damage caused by improper use of the stethoscope, and other expenses not specified here.
3. The guarantee is valid only where this product is officially commercialized by the manufacturer or by local distributor appointed and authorized by the manufacturer. This guarantee and technical assistance are restricted to the Brazilian territory.
4. During the guarantee period the manufacturer will repair the stethoscope in 30 days from the date of arrival of the product to be repaired.
5. The manufacturer reserves the right to replace the defective stethoscope by a new one if it is necessary and this is only by the judgment and decision from the manufacturer.
6. All repairs executed during the guarantee do not extend the deadline of the guarantee.
7. When the guarantee expires the manufacturer will manage to keep the necessary components and to supply technical assistance during 5 years in order to offer paid services to users.
8. All services of maintenance and consumed components by improper use even during the guarantee period will be charged.
9. The stethoscope loses its guarantee by improper handling, direct exposure to sunlight or to extreme temperatures or damages by dropping down, etc. Defects or other damages caused by improper use are excluded from the guarantee; manufacturing problems, please contact our:
Customer Service in Brazil:
São Paulo/ Grande São Paulo: Phone: (11) 3813-1092
Other regions: 0800-555-750