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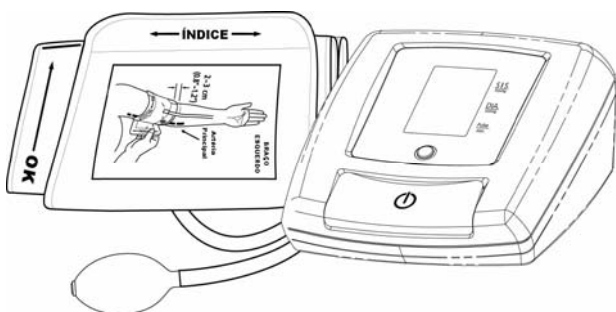
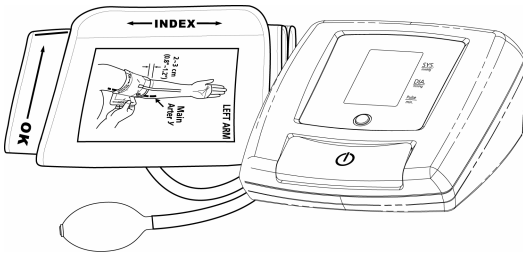
Instruction Manual

Características:

- ✓ Semi-Automatic
- ✓ Easy-to-read Display
- ✓ 60 Automatic Memories
- ✓ Beeping Reminder

CONSULT YOUR PHYSICIAN IF YOU FEEL VARIATION IN YOUR PRESSURE RESULT. THE USE OF THIS PRODUCT DOES NOT DISPENSE THE MEDICAL ADVICE.

DO NOT USE THE PRODUCT FOR INVASIVE MEASUREMENT

Introduction

Blood pressure measurements determined by MG-20 are equivalent to those obtained by a trained observer-using cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard, Electronic or Automated Sphygmomanometers. MG-20 is to be used by adults only for personal use.

Attention: Please read this manual carefully before use. For specific information on your own blood pressure, contact your physician. Please be sure to keep this manual.

How MG-20 works

Techline's MG-20 uses oscillometric method to detect your blood pressure. Before the cuff starts inflating, the device will establish a baseline cuff pressure equivalent to the air pressure. Your MG-20 unit will determine the appropriate inflation level based on one's pressure oscillations, followed by cuff deflation. During the deflation, the device will detect the amplitude and slope of the pressure oscillations and thereby determine the systolic pressure, diastolic pressure, and pulse for you.

About Blood Pressure

What is Blood Pressure?

Blood pressure is the pressure exerted on the artery tube while blood flows through the arteries. The pressure measured when the heart contracts and sends blood out of the heart is systolic (highest). The pressure measured when the heart dilates with blood flowing back into the heart is called diastolic (lowest) blood pressure.

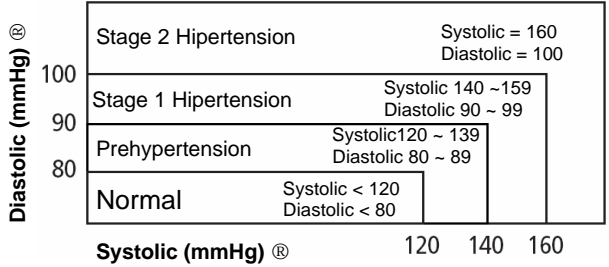
Why Measure Your Blood Pressure?

Among the various health problems afflicting modern people, problems associated with high blood pressure are by far the most common. High blood pressure's dangerously strong correlation with cardiovascular diseases and high morbidity has made measuring blood pressures a necessity of identifying those at risk.

Blood Pressure Standard

The World Health Organization (WHO) and National Blood Pressure Education Program Coordinating Committee have developed a blood pressure standard, according to which areas of flow- and high- risk blood pressure are identified. This standard, however, is a general guideline as individual's blood pressure varies among different people and different age groups, etc.


It is important that you consult with your physician regularly. Your physician will tell you your normal blood pressure range as well as the point at which you will be considerate at risk. For reliable monitoring and reference of blood pressure, it is recommended to keep long term records.


Blood Pressure Fluctuation

Blood pressure fluctuates all the time!

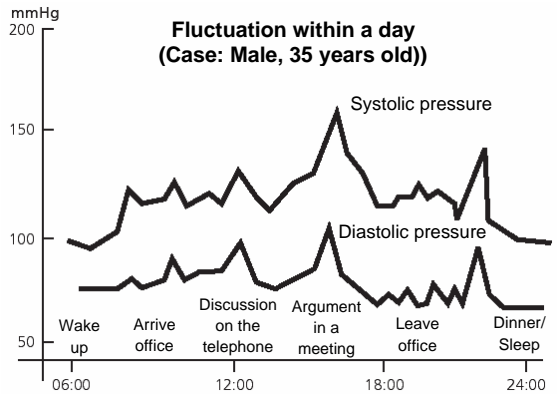You should not be over worried if you encountered two or three readings at high levels. Blood pressure changes over the month and even throughout the day. It is also influenced by season and temperature.


Name/Function of each part

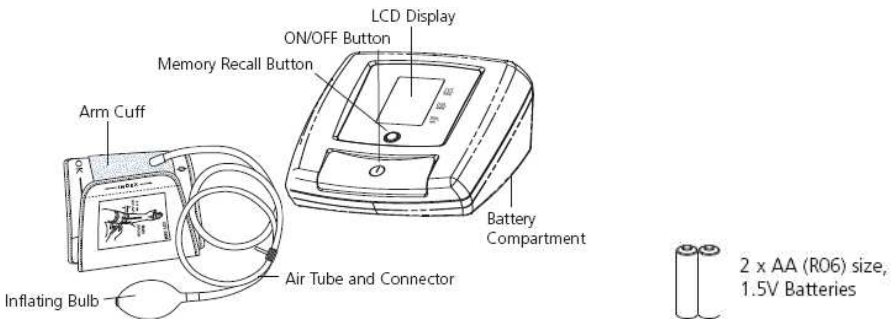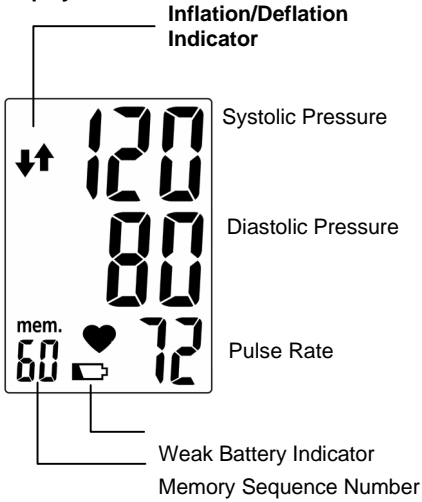Display Explanation

Display:

Symbols:

-  Appears when manual cuff inflation is needed
-  Appears when cuff deflation is needed
-  Appears when batteries should be replaced
-  Shows the pulse rate per minute
-  Occurs when a mistake was made during measurement

Error Messages

EE

Measurement Error: Make sure the T-plug is securely connected to the air socket and measure again quietly. Wrap the cuff correctly and keep arm steady during measurement. If error keeps occurring, contact the service center.

E1

Air Circuit Abnormality: Make sure the T-plug is securely connected to the air socket and measure again quietly. If error keeps occurring, contact the service center.

E2

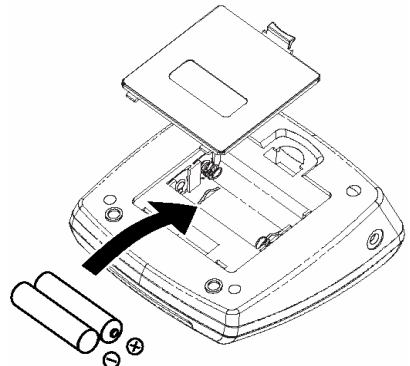
Pressure Exceeding 300 mmHg: Press the air release valve to rapidly exhaust the air in the cuff, switch the unit off, and measure again quietly. If error keeps occurring, contact the service center.

E3

Data error: Remove and reload the batteries. If error keeps occurring, contact the service center.

Installing the Batteries

1. Press down and lift the battery cover in the direction of the arrow to open the battery compartment.
2. Install or replace 2"AA" sized batteries in the battery compartment according to the indications inside the compartment.
3. Replace the battery cover by clicking in the bottom hooks first, then push in the top end of the battery door.

You need to replace the batteries when:

1. Low battery symbol appears on display.
2. The POWER  button is pushed and nothing appears on display.

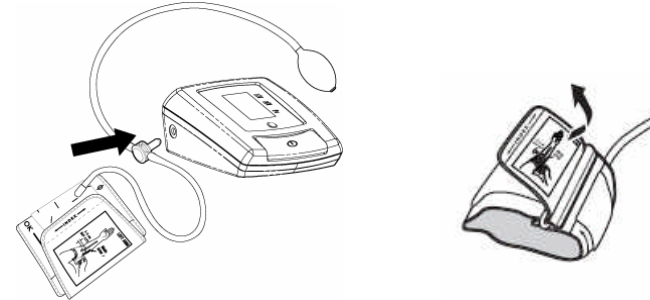
Note:

Batteries are hazardous waste.
Do not dispose them together with the household garbage.

Applying the arm cuff


1. Plug in the cuff connecting tube into the unit.
2. Unwrap the arm cuff, leaving the "Index" end of the cuff through the D-ring of the cuff.

3. Put your left arm through the cuff loop. The "OK" range indication should be positioned closer to yourself with the rubber tube pointing in the direction of your arm. **Position the artery mark (Φ) over the main artery (on the inside of your arm) in the upper arm.**

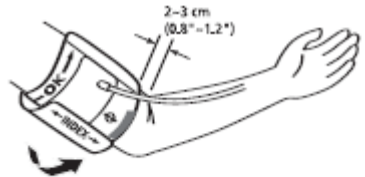4. Turn your left palm upward and place the edge of the arm cuff at approximately 2 to 3 cm above the inner side of the elbow joint. Tighten the cuff by pulling the "Index" end of the cuff.


5. **If the Index line falls within the OK range, indicated at the edge of the cuff, this cuff is suitable for you.** If the Index line falls out of the OK range, you may need a special cuff for measurement.

Taking a Measurement

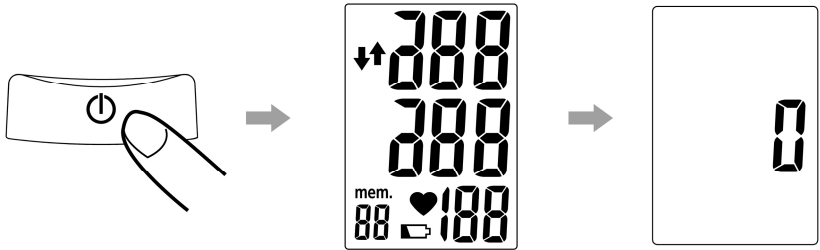
1. Sit down and press the POWER button. All display will appear for approximately one second before returning to "0". The downward arrow "▼" will flash, releasing the air from the system until "0" appears on display.

The unit is ready for measurement at this point.

2. Press the bulb to increase pressure in the arm cuff. The display will show the pressure reading within the cuff.

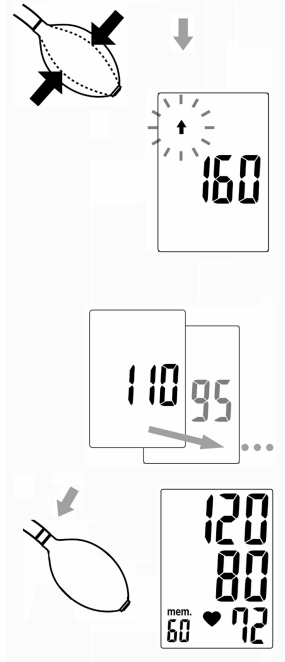Continue pumping until the pressure reaches approximately **30~40 mmHg above your normal systolic pressure**. **Three short beeps will be heard as the pressure reaches 170 mmHg to remind you to stop pumping.** **If the pressure inside the cuff is not enough, upward arrow (▲) will show and continuous short beeps will be heard to remind you to pump up to the pressure of 200 mmHg. You will hear three short beeps to remind you to stop pumping.**

3. Sit still and wait for the monitor to deflate and measure your blood pressure. **It is important to remain still and quiet during measurement otherwise it can affect the results.**
4. After completing the measurement cycle, the monitor will display systolic, diastolic, and pulse readings **at once followed by the sound of a long beep.** Measurement is now completed, and the reading is automatically stored in the monitor.
5. Press the POWER button to turn off the power. If no button is pressed, the unit will shut off automatically in 1 minute.
6. **Measurement readings:** The display will automatically show the systolic and diastolic pressure in mmHg after each measurement and the pulse rate/min too.

Please see the example below:

Systolic	Diastolic	
12.8	7.9	<u>= 13 by 8</u>
—	—	
Rounding out number	Rounding out number	

Memory

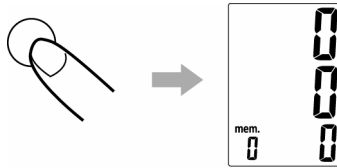
1. To recall stored blood pressure readings from memory, simply press the Memory button. The last set of memorized readings will be displayed.
2. Another press of the Memory button will recall the previous set of readings.
3. All readings stored in memory will be displayed with its sequence number.

Clearing Values from Memory

Press and hold the “M” key for approximately 5 seconds, then the data in the memory zone can be erased automatically.

Press and hold for 5 seconds.

Note: The data in the memory zone can be erased if any of the batteries is removed. Suggest the users to first take record the data before replacing or moving away the batteries.

Helpful Tips

Here are a few helpful tips to help you obtain more accurate readings:

- Do not measure your blood pressure immediately after consuming a large meal. To obtain more accurate readings, please wait one hour before measuring.
- Do not smoke or drink alcohol before measuring your blood pressure.
- It is important that you relax during measurement. Try to take a 20-minute rest before a reading.
- Do not take measurements if you are under stress or under tension.
- Take your blood pressure at normal body temperature. If you are feeling cold or hot, wait a while before taking a measurement.
- If the monitor is stored at very low temperature (near freezing), have it placed at a warm location for at least one hour before using it.
- **Wait about 20 minutes before taking the next pressure measurement.**

Troubleshooting

If any abnormality should arise during use, please check the following points.

Symptoms	Check Points	Correction
No display when the POWER button is pressed	Have the batteries run down?	Replace them with two new batteries.
	Have the batteries' polarities been positioned incorrectly?	Re-insert the batteries in the correct position.
EE mark shown on display or the blood pressure value is displayed excessively low (high)	Is the cuff placed correctly?	Wrap the cuff properly so that it is positioned correctly.
	Is the cuff pressure pumped enough? (30~40 mmHg above normal systolic value?)	Measure it again and pump up 30~40 mmHg more than normal systolic value.
	Did you talk or move during measurement?	Measure it again quietly. Keep arm steady during measurement.
Did you shake the arm with the cuff on?		

Note: If the unit still does not work, please contact the service center. Under no circumstance should you disassemble the unit or repair it.

Cautionary Notes

1. The unit contains high-precision components. Therefore, avoid extreme temperatures, humidity, and direct sunlight. Avoid dropping or strongly shocking the main unit, and protect it from dust.
2. Clean the blood pressure monitor body and the cuff carefully with a slightly damp, soft cloth. Do not wash the cuff or use chemical cleaner on it. Never use thinner, alcohol or petrol (gasoline) as cleaner.
3. Leaky batteries can damage the unit. Remove the batteries when the unit is not used for a long time.
4. The unit should not be operated by children to avoid hazardous situations.
5. If the unit is stored near freezing, allow it to acclimate at room temperature before use.
6. For users diagnosed with common arrhythmia (atrial or ventricular premature beats or atrial fibrillation), diabetes, poor circulation of blood, kidney problems, or for users suffered from stroke, or for unconscious users, the device may have difficulty in determining the proper blood pressure.
7. To stop operation at any time, press the on/off button and the air in the cuff will be rapidly exhausted.

Specifications

Measurement Method	: Oscillometric
Measurement Range	: Pressure: 40~250 mmHg; Pulse: 40~199 beats/ minute
Pressure Sensor	: Semi conductor
Accuracy	: Pressure: ± 3 mmHg; Pulse: $\pm 5\%$ of reading
Inflation	: Manual Inflation
Deflation	: Automatic Pressure Release Valve
Memory capacity	: 60 memories
Auto-shut-off	: 1 minute after last button operation
Operation Environment	: 10°C~40°C (50°F~104°F); 40%~85% RH max.
Storage Environment	: -10°C~60°C (14°F~140°F); 10%~95% RH max.
Power Source	: 3V DC, two R06 (AA) batteries
Dimensions	: 106 (L) X 110 (W) X 50 (H) mm
Weight	: 285 g (G.W) (w/o batteries)
Arm circumference	: Adult: 24~36 cm (9,4"~14,2")
Limited users	: Adult users
K.B.	: Short time operation 2 minutes.

: Type BF
Device and cuff are designed to provide special protection against electrical shocks.

*Specifications are subject to change without notice.

Warranty

1. This product is guaranteed for 2 years from the date of acquisition and this guarantee is effective only by presenting the **RECEIPT OF PURCHASE** stating the buyer's name, date of purchase, name, and address of the distributor.
2. Only defective products with manufacturing problems are covered by this guarantee. Devices with malfunctioning problems caused by misuse or improper handling even if not intentionally, are not covered by guarantee.
3. The guarantee covers the functioning of the monitor with the exception of the cuff because it is a fragile piece and if handled improperly it can blow up or to be punctured, etc.
4. This guarantee does not cover damages by dropping down the device, direct exposition to high humidity, water, battery leakage, and the batteries included in the device, repairs executed by unauthorized person, violation of the device by the customer or by unauthorized person.
5. The guarantee does not cover the cost of round-trip transport for repair, the result or damage caused by improper use of the device and other expenses not specified here.

6. The guarantee is good only in countries where this product is officially commercialized by the manufacturer or the local distributor and authorized by the manufacturer, always with the presentation of the receipt of the purchase.
7. This is a non-transferable guarantee; it means it is good for the first consumer only.
8. During the period of this warranty the producer is obligatory to repair the product 30 days from the date of receiving the device for repair.
9. The manufacturer reserves the right to replace the defective product by a new one in case of need and decision of the manufacturer.
10. All repairs executed during the period of warranty do not extend the deadline of the guarantee.
11. After the guarantee is expired, the manufacturer has to manage to reserve components and to supply technical assistance during 5 years in order to offer paid-services to users.
12. During the warranty period all services of maintenance and of consumed components under bad manipulation are not acceptable.
13. The periodical revision/calibration of the product is not covered by the guarantee.
14. The technical assistance and warranty are restricted to the Brazilian territory only.
15. This guarantee does not cover damages by misuse or improper handling, even if not intentional, dropping down the device, direct exposure to high humidity, water, and sunshine.

Customer Service Center

From Monday to Friday (08:00 a.m. to 06:00 p.m.).

In Brazil: São Paulo and Grande São Paulo: Phone: (11) 3813-1092.

Other regions: 0800-555-750.

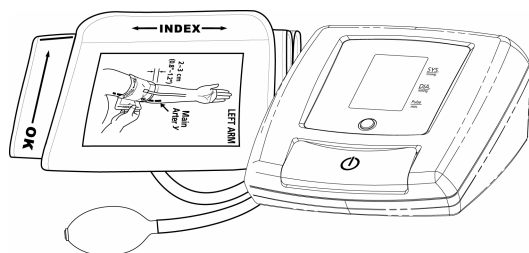
Other countries: please contact your local distributor.

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